

Slide finger on device screen to switch between start screens. That is, to go from the screen on the left above to the screen on the right above, slide finger from right to left. To get back to the screen on the left above, slide finger from left to right.

TESTING

Connect the probe to the red connector. Select either DPOAE Quick: 3 out of 4 button or DPOAE Quick: 4 out of 4 button






Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select the test ear by pressing either the Right or Left button. Select balloon mode to show balloons when the test is finished.




The device will check if the probe fit is correct prior to measurement. If the probe fit is correct (indicated by a green light), the test will start automatically.

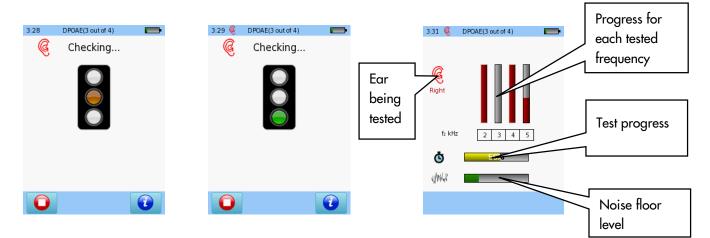




TEST RESULTS (bar graphs)

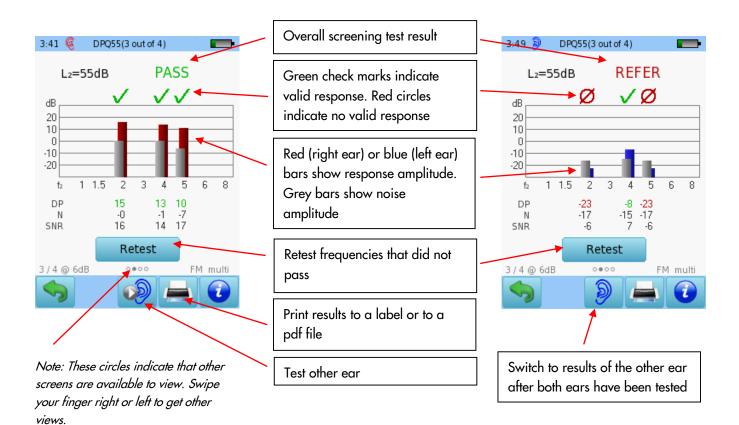



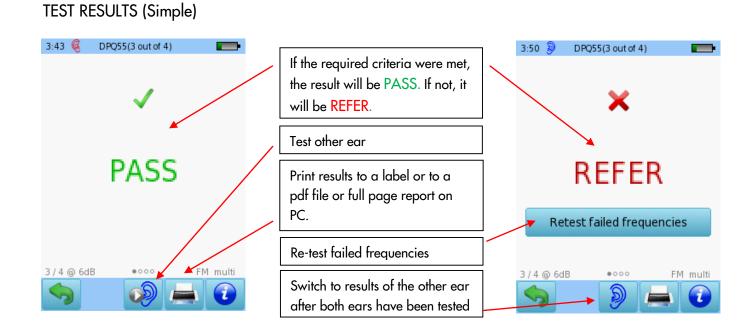



Quick Guide Information

Release date:	2017-01-11
Revision:	02
Valid from:	Firmware Rev. 2.1, Mira PC Software Rev. 1.6.4
Applies to:	Model: SOH100098, web description SOH00

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH medical for further information.

The latest revision of the Quick Start Guide is available online at www.pathme.de/download.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

PATH medical GmbH Landsberger Straße 65 82110 Germering Germany Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: www.pathme.de

