



# QUICK START GUIDE

## SENTIERO DESKTOP DEVICES VIEWING AND PRINTING TEST RESULTS

### HOW TO VIEW PATIENT TEST RESULTS

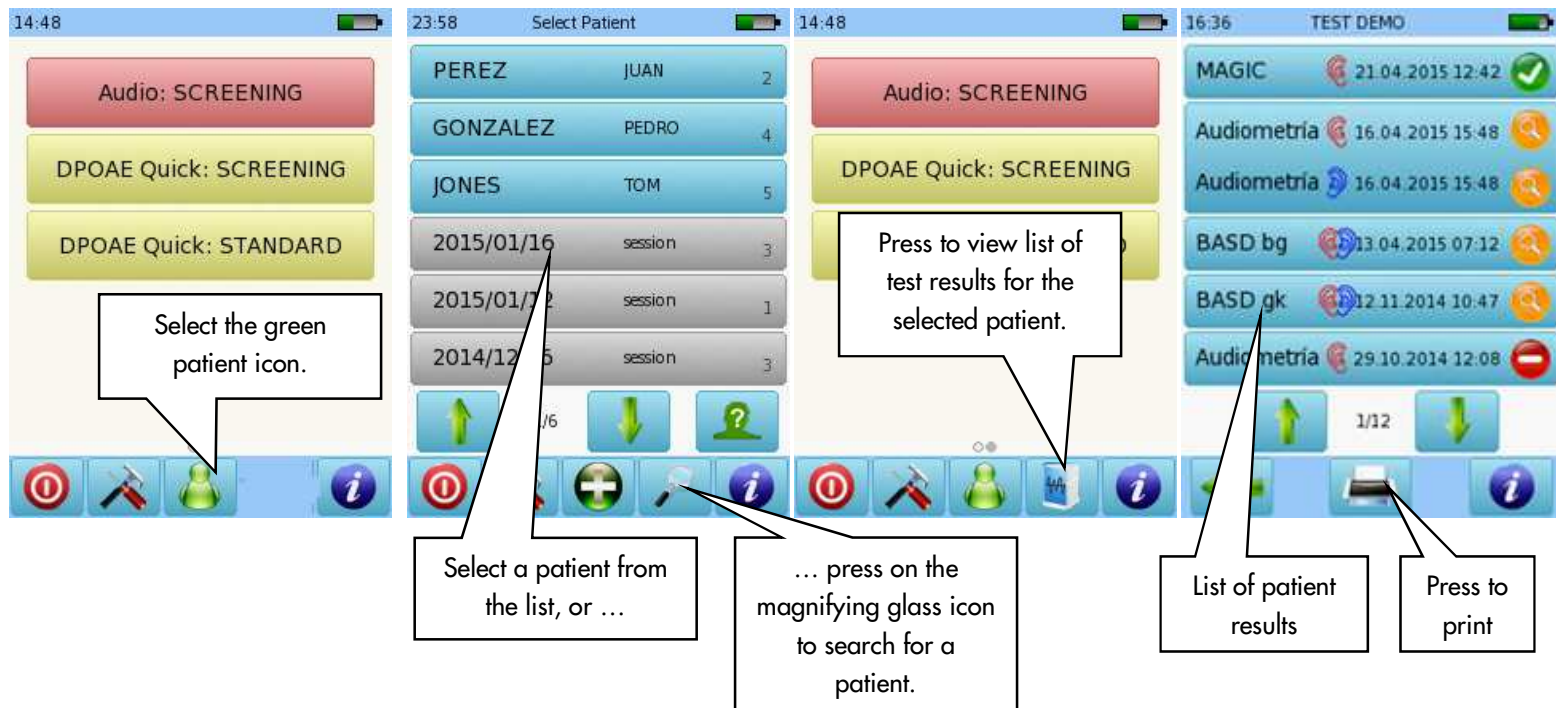

### HOW TO PRINT PATIENT TEST RESULTS

You can select a stored test to print as indicated above. To print a test after you have finished it and saved the data, press the printer icon  at the bottom of the screen. The list of tests performed for the current session will be displayed. You can print the test results as "Session", or enter patient details.

# QUICK START GUIDE

## SENTIERO DESKTOP DEVICES VIEWING AND PRINTING TEST RESULTS

**14:50 Test View**  
 Audio 14:49 [checkmark] [info]  
 Audio 14:49 [checkmark] [info]

**14:51 Patient details**  
 Do you want to enter patient details before printing?  
 ID: [Press to edit]  
 First name: [session]  
 Last name: [ ]  
 Date of Birth: 2016/01/21

**4:57 Patient details**  
 RENAMED [X]  
 [QWERTY keyboard]

**4:57 Patient details\***  
 Do you want to enter patient details before printing?  
 ID: [Press to edit]  
 First name: [RENAMED]  
 Last name: [ ]  
 Date of Birth: 11/02/2015

**Callout boxes:**

- Select which tests you want to print.
- Press this button to clear all check marks.
- Press OK when test selection is complete
- You may enter patient details before printing
- Press OK to print as Session, or ...
- Enter patient details using keyboard
- Press OK to print with patient name

**14:53 Test View**  
 Audio 14:49 [checkmark] [info]  
 Printer Error  
 No label printer connected  
 OK

**Callout box:**  
 An error message will appear if the device is not connected to a label printer or a computer with MIRA Direct Print (see below for details)



# QUICK START GUIDE

## SENTIERO DESKTOP DEVICES VIEWING AND PRINTING TEST RESULTS

### PRINTING USING LABEL PRINTER



Connect the label printer cable to the printer and to the corresponding connector on the device. Make sure both the printer and the device are plugged into the power source.



Your label(s) will print (one or more, depending on how many tests you have selected).

### PRINTING USING MIRA DIRECT PRINT




Install the MIRA software, which includes Direct Print utility, on a PC. This will install this icon on your desktop: Double click to launch Direct Print. The icon will appear on your taskbar to indicate it is running in the background and available to print. If you open MIRA software, Direct Print is paused. Close MIRA and Direct Print will resume.


Connect the USB cable to the PC and to the USB port on the Sentiero device.



A preview of the printout will open on the PC. Select File – Export Document – PDF file.

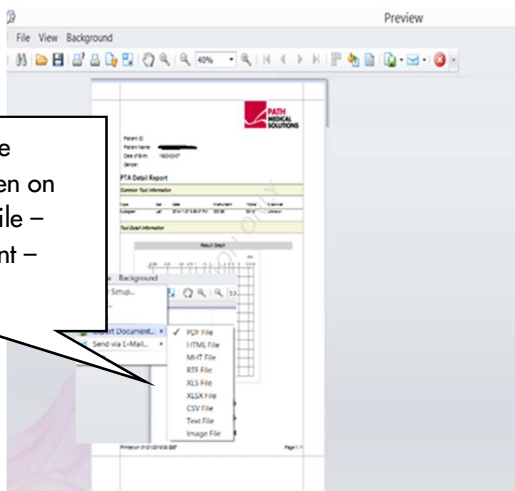

“PDF Export Options” window opens. Click OK. “Save as” window opens. Change the document name if desired. Select the location where you would like to save it, and press Save.

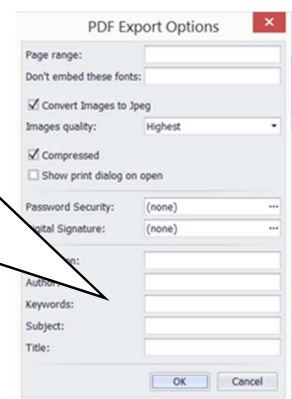

# QUICK START GUIDE

## SENTIERO DESKTOP DEVICES VIEWING AND PRINTING TEST RESULTS

---

### Quick Guide Information

Release date: 2017-01-12  
Revision: 02  
Valid from: Firmware Rev. 2.1, Mira PC Software Rev. 1.6.4  
Applies to: Model: SOD100497, all web descriptions.

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH medical for further information.

The latest revision of the Quick Start Guide is available online at [www.pathme.de/download](http://www.pathme.de/download).

Errors and omissions excepted.

### Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

PATH medical GmbH  
Landsberger Straße 65  
82110 Germering  
Germany

Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: [www.pathme.de](http://www.pathme.de)

