
### TURNING ON/OFF, CHARGING THE DEVICE





### START MENU

Note: this screen (test options) can vary depending on your set up

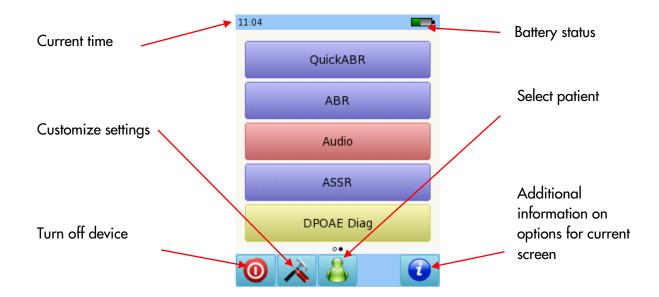



### **TESTING**

### AUDITORY BRAINSTEM RESPONSE (ABR) SCREENING

Please note that single ear testing is shown for ABR screening. For binaural testing instructions, please refer to the ABR diagnostic Quick Start Guide


Connect the electrode cable connector.



to the white connector and the probe to the red



Clean the patient's skin and place the electrode pads. Connect the clips to the corresponding sites. Recommended montage is:

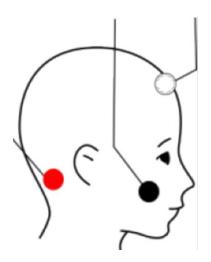

White: high forehead

Black: cheek Red: mastoid



Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select QuickABR.

Select the test ear by pressing either the Right or Left button.



A calibration will be performed prior to measurement. A green light indicates successful calibration.

Also, an impedance check will be performed prior to measurement. An impedance below 6 k ohm is recommended for best test results.


After successful ear probe calibration and impedance check, the test will start automatically.

If yellow or red lights are displayed, check to make sure you are using the correct probe tip and ear tip size, that the the eartip in inserted correctly, and that there is no excessive ambient noise.






#### **ABR SREENING TEST SCREEN**

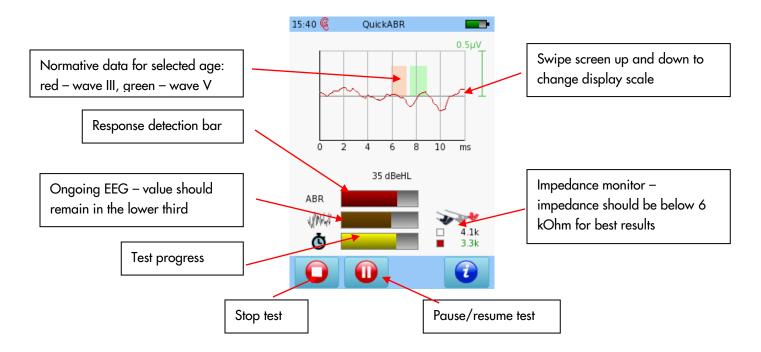

#### ABR SREENING TEST RESULTS

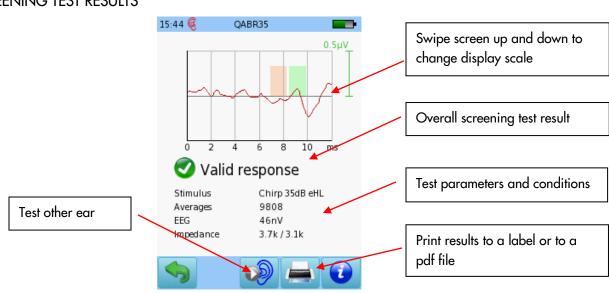



#### **Quick Guide Information**

Release date: 2016-12-22

Revision: 01

Valid from: Firmware Rev. 2.1, Mira PC Software Rev. 1.6.4

Applies to: Model: SOH100360, web descriptions SOH06, SOH07, SOH08 and SOH09

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH MEDICAL reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH MEDICAL for further information.

The latest revision of the Quick Start Guide is available online at www.pathme.de/download.

Errors and omissions excepted.

#### **Copyright Notice**

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH MEDICAL GmbH.

Copyright © 2016 PATH MEDICAL GmbH

PATH MEDICAL GmbH Landsberger Straße 65 82110 Germering Germany

Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: www.pathme.de

0124

