

# QUICK START GUIDE

## QSCREEN DIRECT PRINT

PATH MEDICAL GmbH  
Landsberger Straße 65  
82110 Germering / Germany  
Tel +49 89 800 76 502 / Fax +49 89 800 76 503  
Info @pathme.de / www.pathme.de

THE SOUND OF SCIENCE



# Quick Guide: QSCREEN DIRECT PRINT

## 1. Required equipment's for QSCREEN direct print

Refer to manual for different printer options.



QSCREEN Docking Station



Seiko SLP 650 label printer



Supported colour laser printer (e.g. brother HL-L3240CDW)



Docking Station Connectors




Seiko printer connectors



Laser Printer connectors

## 2. Power on the devices & connection to the printer

- Press power button to turn on the QSCREEN.
- Make sure the **printer and Docking Station** are connected to the **mains power**.
- Ensure that the Bluetooth icon is available in the header of the QSCREEN display.

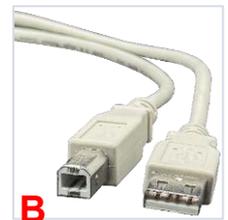

B

A

### Docking Station to Printer


- Connect via **USB A/B** cable - plug.
- Type A end into the Docking Station USB port (1).
- Type B end into the label printer (2) or laser printer (3).

## 3. Printing methods & navigation

### 3.1. Immediate post-test printing

After test completion, **Printer** button appears at the bottom of the screen.

- Click the **Print** button.
- Print to connected printer (label or laser).




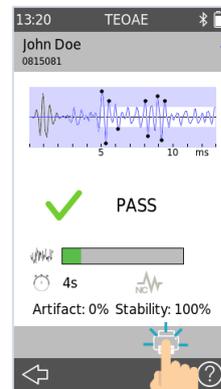


### 3.2. From the patient list

From **Home** screen, press the **Print** button. The patient list displays. Select the desired patient.

### Single Report


- Open the test result to be printed.
- Press **Print** button in the detailed view.
- Detail report** is being printed.

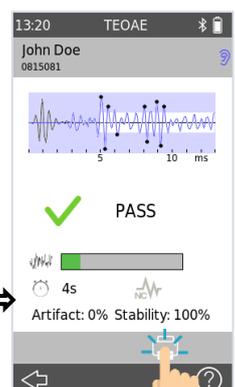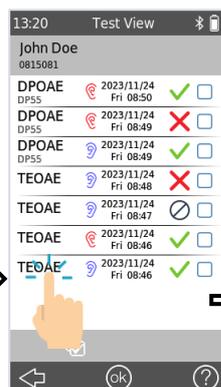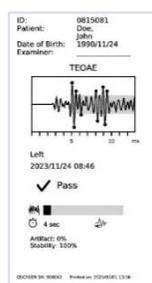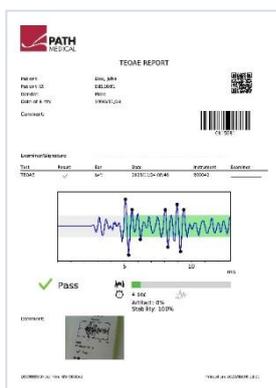

# Quick Guide: QSCREEN DIRECT PRINT



## Summary Report

- Select the desired patient.
- Use the checkboxes next to the test result.
- Select multiple tests for printing.
- Press OK button.
- **Summary report** is being printed.

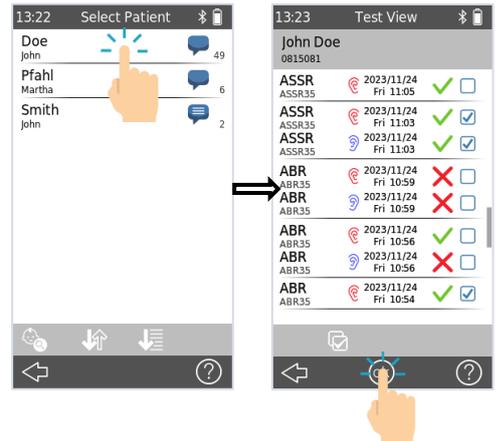

## Side by side Report


- Select the desired patient.
- Check the desired test results for both Left and Right ear.
- Hold the OK button (don't just press).
- Wait for the printing message box to appear.
- The **side-by-side report** is being printed with left and right ear results.

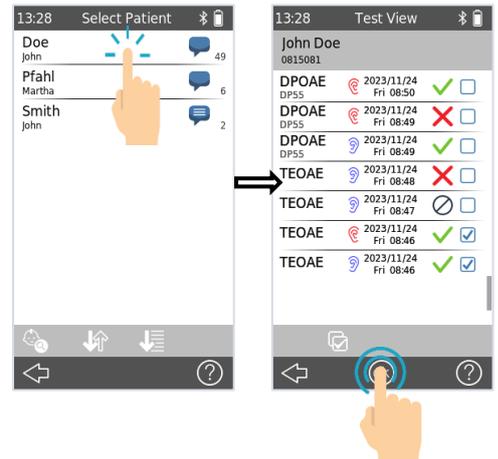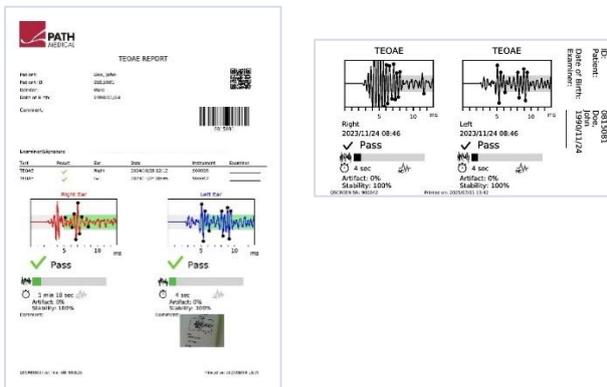

## 3.3. Patient Test Menu Access

From **Home** screen, press **Select Patient** and choose the desired patient from the list.

- **Detailed Single Test (1)**: Press specific result button first, then print.
- **Summary Report (2)**: Press **Print** button for last/best results. For the given example, 8 pass results will be printed irrespective of the tests done.
- **Side-by-side (3)**: Hold **Print** button until message box appears. For the given example, 4 labels are printed, one from each protocol regardless of tests made.

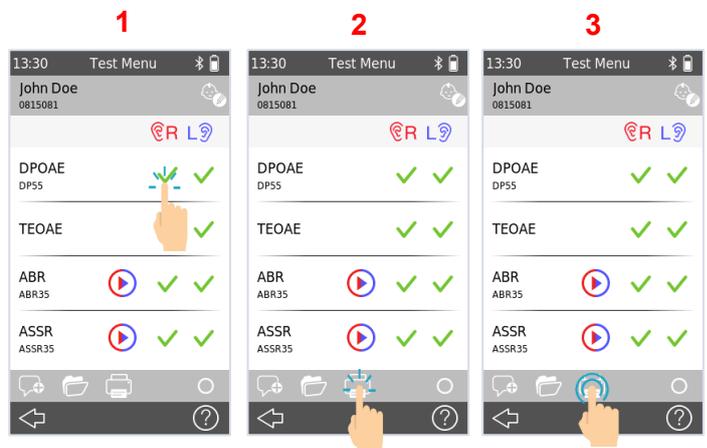

# Quick Guide: QSCREEN DIRECT PRINT

Release date: 2026-02  
File name: 100800-44\_Quick\_Guide\_QScreen\_Direct Print  
Revision: 01  
Valid from: Firmware Revision 2.2  
Applies to: QSCREEN

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH MEDICAL reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH MEDICAL for further information.

Errors and omissions excepted.

## Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH MEDICAL GmbH.

Copyright © 2026 PATH MEDICAL GmbH

Tel.: +49 89 800 76 502

Fax: +49 89 800 76 503

PATH MEDICAL GmbH  
Landsberger Straße 65  
82110 Germering  
Germany  
Internet: [www.pathme.de](http://www.pathme.de)

  
0124