

December 07, 2020

Dear customer,

PATH MEDICAL is pleased to announce the release of new Senti and Sentiero device firmware (2.7 and 2.7.1) and Mira PC software (2.3.0 – 2.3.2).

The main updates are described below. Please pay particular attention to the important information described in chapter 3 (“Important Notices”).

1. FIRMWARE MODULES AND FEATURE UPDATES

The following new features are now available with Senti/Sentiero firmware:

- **ABR:**
Extended stimulus level range of up to 100 dB nHL for tone-bursts with insert ear-phones PIEP (order no.: 100860)
- **ASSR:**
Quick ASSR test module (order no.: 100465-Quick) providing a pass/refer result at a selectable stimulus level
- **αEPS:**
Quick ABR / Quick ASSR test sequence (order no.: 101202) can be started via an extra button in the “AEP” section of the device test modules
- **Miscellaneous:**
 - In “AEP preferences” stimulus level unit changeable (dB nHL or dB eHL) for Quick ABR and Quick ASSR
 - Bug fixes (Ukrainian keyboard, ECoChG marker movement, ASSR settings, pathTrack, ABR result view, Noah communication)

2. PC SOFTWARE

- **Mira – Reporting, printout:**
 - Naming convention for pdf printout file name can be configured
 - Age group for normative data shown in ABR report
 - Bug fixes (Remote control with USB 3.1, more accurate placement of ABR markers)

3. IMPORTANT NOTICES

Please read the following important information carefully and make sure to inform your customers about this information.



PATH MEDICAL STRONGLY RECOMMENDS TO USE THE LATEST MIRA AND THE LATEST PATH SERVICE TOOL INCLUDING THE LATEST SPEECH SAMPLES AND FIRMWARE AND TO UPDATE ALL DEVICES WITH THE LATEST FIRMWARE BY USING MIRA OR THE PATH SERVICE TOOL UPDATE FUNCTION. PLEASE NOTE THAT SOME DEVICE TYPES ARE NOT RECOGNIZED BY OLDER MIRA AND PATH SERVICE TOOL VERSIONS.



WHEN UPDATING DEVICE FIRMWARE, THE TEST MODULE PARAMETERS ON THE DEVICE MAY BE RESET TO FACTORY PRESETS UNDER SPECIAL CIRCUMSTANCES (E.G. WHEN UPDATING DEVICES WITH FIRMWARE 1.6.4485 OR LOWER), I.E. INDIVIDUALLY SET TEST MODULE PARAMETERS MAY BE OVERWRITTEN. THEREFORE, PLEASE MAKE SURE TO KEEP TRACK OF INDIVIDUAL PARAMETERS BEFORE UPDATING THE FIRMWARE.

4. GENERAL INFORMATION

For service requests (i.e., technical or handling problems) regarding Senti and Sentiero hardware, firmware or software and related accessories and service tools please contact PATH MEDICAL service at service@pathme.de.

For further information about PATH MEDICAL products please refer to <http://www.pathme.de> or contact info@pathme.de.
