

March 16, 2026

Dear customer,

PATH MEDICAL is pleased to announce the release* of new nanoAudio software (1.0.0.977) and device firmware (build 964).

NANOAUDIO FEATURE UPDATES AND FIXES

The following new features/fixes will be available for nanoAudio with the update:

- **Improvements to pure tone audiometry and workflow**

- Pulsed tone stimuli are now supported. Each nanoAudio protocol can be individually configured to perform audiometric testing with continuous tones or pulsed tones.
- The audiograms now support the 'no response' symbol which can be set if the patient did not respond to the highest level which was tested.
- When using the keyboard to control manual audiometry, the level control direction (arrow up, arrow down to increase or decrease intensity level) can now be configured.
- The number of protocols which are available for individual configuration has been increased from 2 to 4.

- **Additional languages**

- French, Spanish and Portuguese translations were added.

- **Report Generation**

- The patient ID now appears on the report.
- Lärm I and Lärm II forms according to DGUV Germany are available and automatically populated with audiogram data and patient demographics.

**Please note that regulatory requirements in your territory may result in delays to this update, as notification and clearance may be necessary.*