

November 13, 2024

**Dear customer,**

PATH MEDICAL is pleased to announce the release of QLINK PC software (1.4.3).

The main updates are described below. Please pay particular attention to the important information described in chapter 3 ("Important Notices").

## **1. PC SOFTWARE**

- **QLINK - HEARING SCREENING TRACKING LGL BAVARIA:**

Added additional configuration options for tracking solution LGL Bavaria:

- The system allows to configure the patient record number in NHS Germany format
- The administrator has the option to protect the export file by setting it to read-only mode

## **2. IMPORTANT NOTICES**

Please read the following important information carefully and make sure to inform your customers about this information.



PATH MEDICAL strongly recommends to **use the latest QLINK** and the **latest PATH Service Tool** including the new firmware. Please update all devices to the latest firmware by using QLINK or the PATH Service Tool update function.



When updating the device firmware, **QLINK will also update the Docking Station software**. This may take **up to 30 minutes**. The QSCREEN turns off during the update process and the Docking Station's LED changes the color several times. **Do NOT turn on the QSCREEN until the update is finished** (see left sidebar in QLINK). Wait until the LED of the Docking Station illuminates white.

### **3. GENERAL INFORMATION**

---

*For service requests (i.e., technical or handling problems) regarding QSCREEN hardware, firmware or software and related accessories and service tools please contact PATH MEDICAL service at [service@pathme.de](mailto:service@pathme.de).*

*For further information about PATH MEDICAL products please refer to <http://www.pathme.de> or contact [info@pathme.de](mailto:info@pathme.de).*

---